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*English translation on next page

Product Description

Our 3D printed prosthetic covers enable you to incorporate your personality into your prosthesis. Your cosmetic cover imitates the anatomical shape and symmetry of your sound leg while offering you different design options and colour combinations. Our team includes certified Prosthetists that have mastered the art of 3D modelling and therefore each cover is designed while having in mind all the clinical demands a prosthetic user may have. Covers are available for most ankle and knee components (Trans-tibial, Trans-femoral, Bilateral).

Our covers come in front and back part, the front part attaches to the prosthesis's pylon and the back part connects to the front part using either strong magnets or screws (suitable for water-based activities). The magnetic attachment takes just seconds!

Each cover is fully custom tailored to the user's prosthetic componentry, by making the appropriate adjustments to the design we allow access to essential components such as charging. We manufacture the covers within our facilities using high-end 3D printers with lightweight, durable and waterproof materials (PETG, NYLON or TPU); using 3D printers enable us to produce with minimal material waste while being 100% recyclable and environmentally friendly. Lastly, each cover is post-processed by hand and painted using automotive techniques and materials to get the best quality finish.

The durability of our covers depends on the user and on how are taken care of. Our covers protect the internal prosthetic components from damage caused by impacts of everyday life, but like any other protective and aesthetic devices, paint may be damaged and the material can break when exposed to impact shocks, scratches and falls. Covers with more perforations tend to be more delicate and detailed, but also very lightweight. Whereas designs with less perforations are more durable but are a bit heavier.

Key points

- Customized to each prosthetic limb.
- Protects your expensive prosthetic components.
 - Magnetic attachment.
 - Water friendly.
- Easy access for the prosthetist when it comes to prosthetic alignment changes or part replacement.
- Easy to clean, smooth surface.
- Countless design options as you also have the ability to choose your own custom design.
 - Light weight.
 - Environmental friendly.

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Caution!

- Don't use any strong chemicals when cleaning your cover as it might affect paint. *Use clean water
 - Don't kneel onto hard surfaces.
- High impacts may cause damage to the cover.

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Step 2:

After you placed the front cover correctly (as seen in the picture), attach the second part of the clamp using 2 of the screws included in the kit.

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
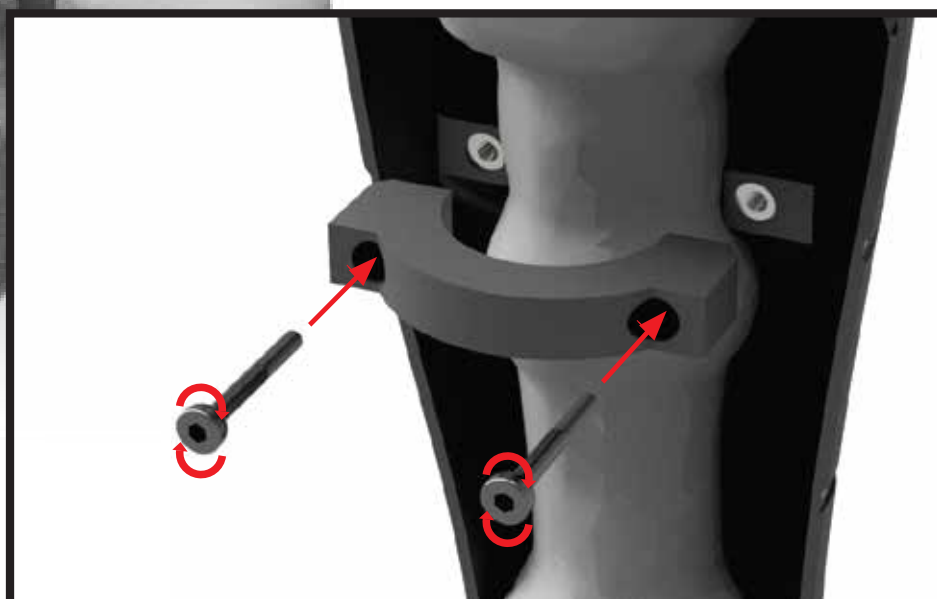
Step 2.1:

At this stage, the front cover needs to be well secured on the prosthetic pole.

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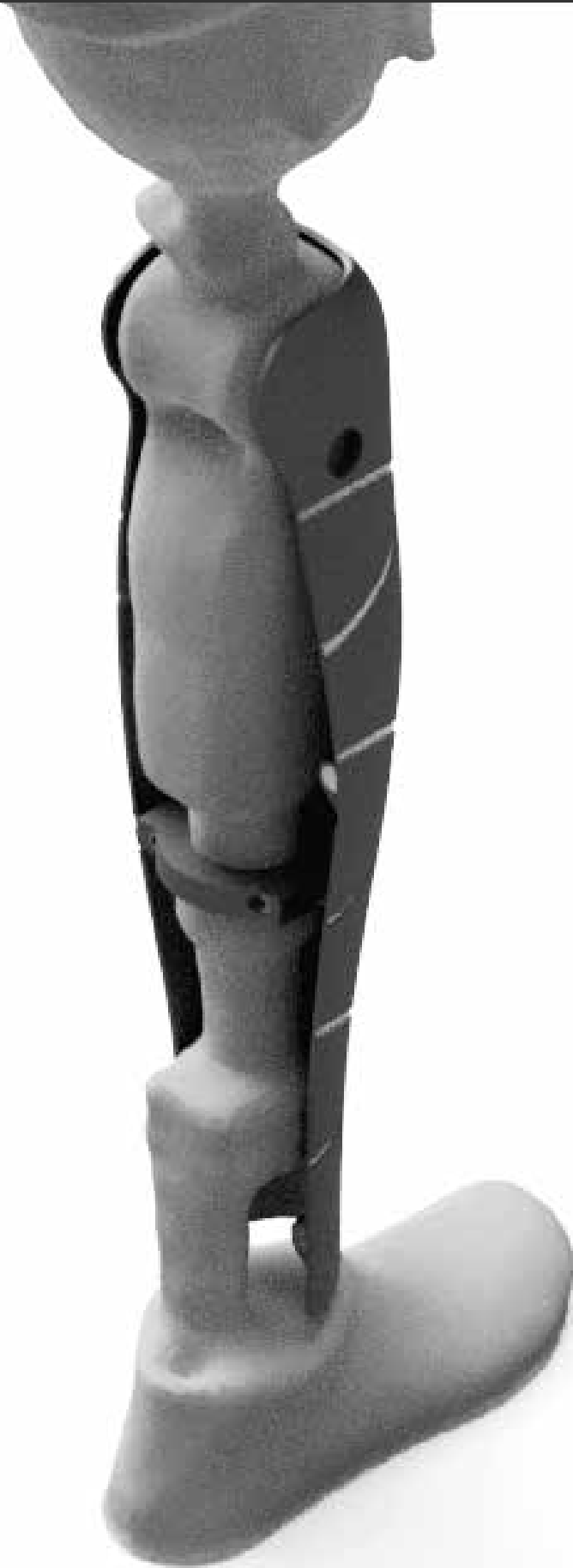
Step 3:

At this point, you just attach the back cover to the front cover!

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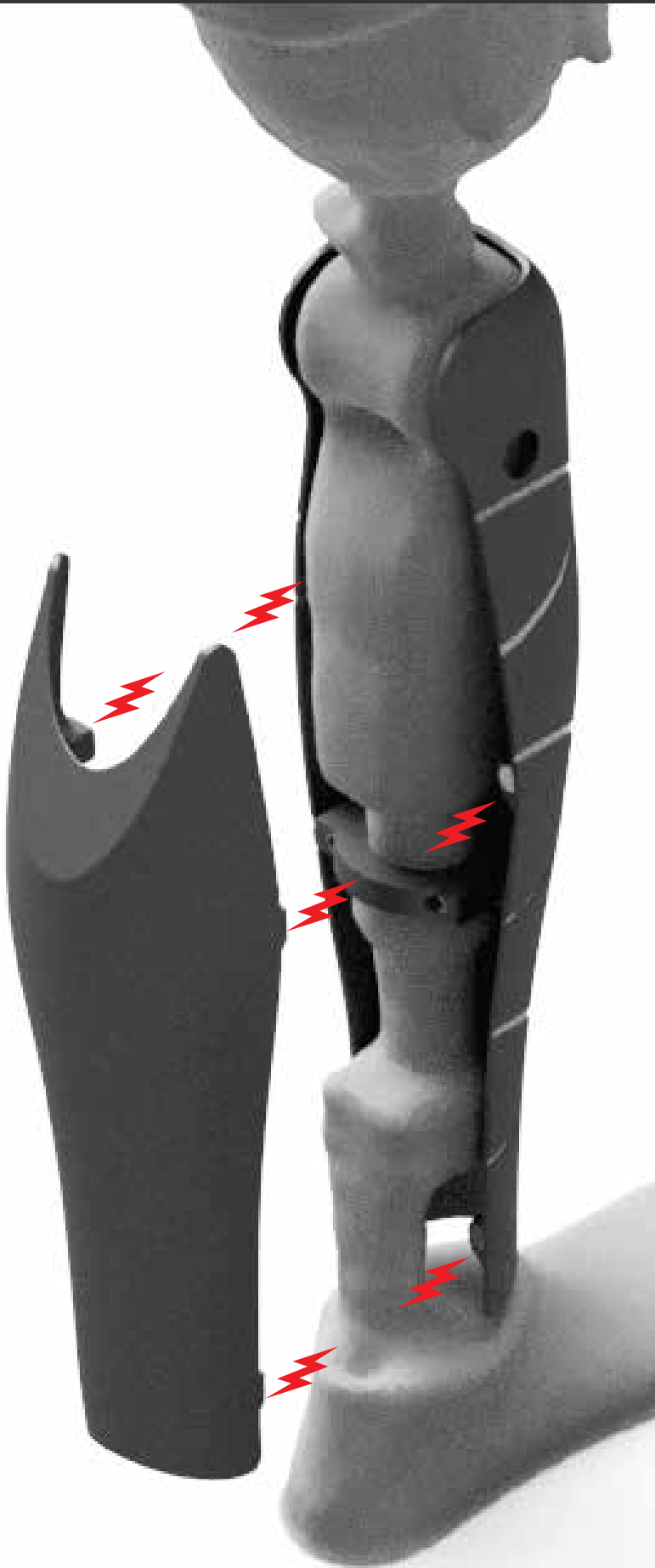Warranty

Our warranty starts from the date of delivery and lasts for a period of one year. The 2 parts and components of the prosthetic cover must be free from defects in material, production and construction when received.

The warranty does not apply to any defect in the prosthetic cover resulting from:

1. Damage due to fitting mistakes.
2. Intentional damage, abnormal storage or environmental conditions or damage from an accident, negligence on your part or on the part of any third party.
3. Any modification or repair by the user or a third party.

All approved warranty claims will be processed and shipped within 21 days of approval.

OrthoDesigns reserves the right to refuse warranty claims at their discretion. All warranty refusals will include a written response outlining the reason.

RETURNS & REFUNDS

OrthoDesigns does NOT issue credit or refunds.

Our prosthetic covers are custom made based on user specifications making it impossible to resell products if they were to be returned. All warranty replacements must have the same design, anatomical shape and finish as the original prosthetic cover

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